



BA-Nr.: 02.05.012/2021-09-A01



REBOTEC® Rehabilitationsmittel GmbH
DE-49610 Quakenbrück, Artlandstr. 57- 59
Telephone: +49 (0) 5431/9416-
Fax: +49 (0) 5431/9416- 66
Web: www.rebotec.de
Email: info@rebotec.de



User Manual

Rollators

	FIXI	FOX
REF	186.00.1X	186.00.2X 186.50.2X
	100 kg	100 kg

X=Variant



Figure similar



Read this User Manual **before using the device for the first time** and keep it at hand for the user as well as for future use and subsequent sales.

**Table of contents**

1. Product description	3
1.1. Introduction	3
2. Features	3
2.1. Intended purpose	3
2.2. Conditions of use	3
2.3. Storage & transport.....	3
2.4. Equipment & functions	3
2.5. Packaging	4
2.6. Disposal	4
3. Symbols & nameplates	4
4. Safety instructions	4
5. Incident reporting.....	5
6. Warranty.....	5
6.1. Complaint.....	5
7. Reuse, lifetime	5
8. Scope of delivery.....	6
9. Setting up for use.....	6
9.1. Unfolding.....	6
9.2. Adjusting the handle height.....	6
10. Product data	6
11. Operation of the product	6
11.1. Prepare for operation.....	7
11.2. Sitting down and standing up.....	7
11.3. Pushing the product	7
11.4. Braking system.....	7
11.5. Folding of the product	7
11.6. Adjusting the brakes.....	7
12. Troubleshooting	8
13. Care and maintenance	8
13.1. Cleaning and care	8
13.2. Maintenance.....	8
-(cracks, breakage, loose parts).....	8
14. Spare parts & accessories	10
15. Drawing & dimensions	11

1. Product description

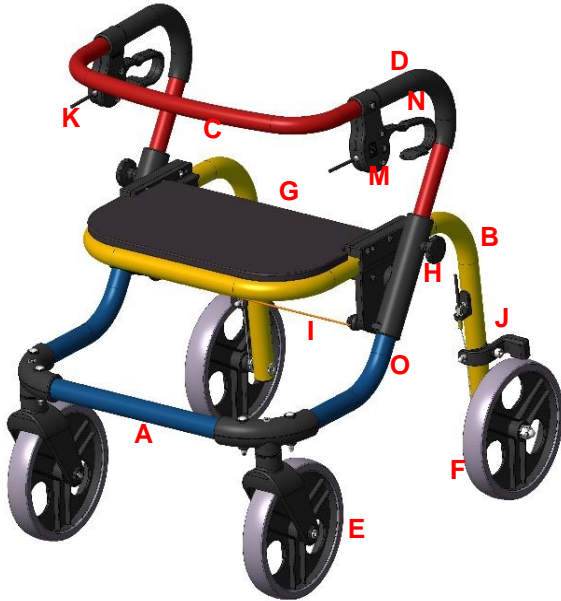


Figure shows:

A	Front wheel bracket
B	Rear wheel bracket
C	Handlebar
D	Handles
E	Front wheel
F	Rear wheel
G	Seat board
H	Star knob nuts, height adjustment
I	Pull cord, folding mechanism
J	Brake
K	Bowden cables
M	Handbrake, lockable
N	Brake lever
O	Nameplate

1.1. Introduction

- The product described here is a medical device of risk class I according to **(EU) 2017/745 MDR**.
- In this user manual, you will find information on the safe and proper initial operation, use, maintenance and care of this product.
- Carefully read this user manual before using the device. Follow the safety instructions and act accordingly.

2. Features

2.1. Intended purpose

- The product is used for support and relief in case of restrictions of the lower extremities. It allows the user to pursue their desire for mobility. There must

be sufficient upper body musculature for safe use. Persons with sensory impairments may not be able to use this item. The article does not impose any special requirements on the education of the user.

Specification:

- Maximum user weight: 100 kg
- Height adjustable handle
- Foldable
- Indoor & outdoor use

Indication

- Impairment of mobility in the case of damage to locomotive/balance system with sufficiently preserved walking ability/coordination, if the use of other walking aids is insufficient
- To maintain/promote/secure walking; additional support for everyday activities:

Transport function: Rollator Fixi / Fox

Sitting down when exhausted: Fixi rollator for children, colorful with seat board, Fox rollator for youth and women, colorful, with seat board, Fox rollator for youth and women, blue, with seat board

- If used indoors, care must be taken to ensure sufficiently spacious living conditions.

2.2. Conditions of use

- Temperature range: -10°C to 35°C; direct radiant heat on the product must be avoided.
- Relative humidity: 40% to 85%. Outside of these conditions, the product must only be used for a short time.
- Folding for transport or storage is possible.
- This product may not be used to transport heavy objects.
- There must be a flat surface of sufficient size for the hoist to be placed on. See 15.

2.3. Storage & transport

- Temperature range: 5 °C to 25 °C; direct radiant heat on the product must be avoided.
- Store the product in dry rooms with a humidity of up to 65%.

2.4. Equipment & functions

- Foldable
- Handle height adjustable (see 9.2 & 15)
- Foldable seat board
- Hand brakes with locking function

2.5. Packaging

- We recommend you keep the packaging for a later transport.
- Dimensions of FIXI: 645mm x 470mm x 675mm
- Dimensions of FOX: 700mm x 465mm x 745mm

2.6. Disposal

- Disposal and recycling of used products and packaging must be carried out in accordance with currently valid regulations. Please contact a disposal company for further information.
- The Infection Protection Act must be observed.
- Please observe any labels and information on packaging material and act accordingly.

3. Symbols & nameplates

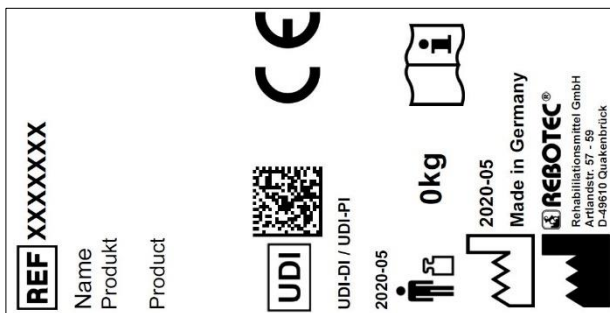


Figure similar.



The nameplate must not be removed!

For ordering spare parts and processing complaints, the device name, serial number and year of manufacture will be required.

Symbol	Meaning	Symbol	Meaning
	Read the manual!		Year of manufacture of Device
SN	Serial number		<i>Important information!</i>
REF	Article number		<i>Maximum user weight in kg</i>
	Care instructions		Maintenance instructions
	Applicable for indoor use only.		Manufacturer
	Visual signal	EN	Language

Symbol	Meaning	Symbol	Meaning
	Read the User Manual!		Do not dispose of as household waste.
CE	Product complies with (EU) 2017/745 MDR		
	Caution! Observe the safety instructions		
UDI	Unique Device Identifier (Data: GTIN / LOT (SN) / year of manufacture)		
MD	Medical device label This product is a medical device.		

4. Safety instructions



Please observe the following safety instructions:

- Before using the device for the first time, the User Manual must be read completely in order to avoid hazards or damage caused by maloperation.
- Before each use, check the device for visible damage or deficiencies.
- In case of malfunctions or damage of any kind, the device must not be used.

Caution: Danger of falling!

- Be sure to check the following aspects before use:
 - Stability of the product.
 - Function of the brakes.
 - Proper locking of the folding mechanism.
 - Tight fit of the star knob nuts.
- Do not carry the product with the patient seated.
- Only use the product if completely assembled.
- Do not slump carelessly on the product.
- Only use the height adjustment in the holes provided for it.
- Avoid using the product on slippery or uneven ground.
- Make sure that the brakes are always in locked position before using the article. Only release the two brakes after you see you are able to stand stably.
- Do not start walking or pushing until you are standing upright and have grasped the handles with both hands.



- Always walk within the product frame and do not push it too far in front of you, otherwise there is an increased risk of falling due to incorrect loading.
- Depending on the health condition, sitting down and standing up by a patient should be done under supervision or must be done with the help of a trained caregiver.
- Users who are prone to uncontrolled actions or movements must be supervised when using the product.
- Push the article according to your mobility. If fatigue sets in while walking, sit on the seat board. Only use the seat board when the brakes are locked!

Caution: Danger of slipping!

- Do not grasp the handles with wet hands, as you could slip and lose your grip.
- When sitting down on the seat board, do not sit on the front edge of the seat, but on the entire seat surface. Before sitting down, bring the brakes into standing position!

Caution: Danger of tipping over!

- When using the item outdoors, walk on level paths only and avoid high curbs as much as possible. If necessary, look for an area with lowered curbs or ask for help getting over the obstacle.
- Avoid walking on steep inclines or declines; otherwise you could lose control of the rolling product.
- Do not use the product on stairs or steps.
- Do not hang any objects on the product.
- Do not lean too far over the handlebar.
- Avoid excessively leaning over the product.

Caution: Danger of breakage!

- Observe the permitted user weight (2.1).
- The product is not a means of transport for heavy objects.
- Do not climb onto or stand on the product.
- The product is not suitable for passenger transport.
- Do not expose the product to great temperature fluctuations (2.2 & 2.3).

Caution: Danger of pinching or catching!

- There is a risk of pinching or catching on all moving parts of the product. Be especially careful when unfolding and folding that no body parts or objects are caught between the moving parts.

Caution: Risk of injury!

- There is a risk of injury with surface temperatures > 40°C. Therefore, do not expose the product to high temperatures (e.g. sunlight, radiators, hot water). Allow the product to cool down before use.



If the product is used by patients with insensitive skin (no temperature perception or skin damage), a heat check (e.g. touching with the back of the hand) must be carried out by the nursing staff.

5. Incident reporting

- Serious incidents in connection with the product must be reported immediately to the manufacturer and BfArM.
 - BfArM: www.bfarm.de/DE
 - Use the reporting functions and provided forms.
 - Manufacturer: pms@rebotec.de

6. Warranty

- For this product the manufacturer grants a warranty of 12 months. Compliance with the general terms and conditions (www.rebotec.de/agbs), as well as the intended use, are a basic prerequisite. The statutory warranty provisions shall apply.
- Unauthorised modifications to this product will void the product's conformity and the warranty.

6.1. Complaint

- Please contact us before returning the product.
- To reduce transport damage, use the original packaging if possible.
- The Infection Protection Act must be complied with.
 - Please make sure that the product does not pose a risk of infection when returned.
 - Enclose the supplied information sheet on safety with the product.
 - Costs may incur for returns that are not marked non-objectionable.

7. Reuse, lifetime

- The product is suitable for reuse. How often the product can be reused depends on the frequency and nature of use.
- Before reuse, the product must be reprocessed hygienically and the technical and functional safety and reliability of the product must be checked and restored if necessary. See 13.

- o For more detailed information, please see the reprocessing instructions (02.12.155) and the maintenance instructions (02.11.149).
- o The documents can be downloaded from the following link.
 - www.rebotec.de/downloads
- When used as intended, the product has a lifetime of 3 years. The effective lifetime may vary depending on the frequency and intensity of use and the general condition.

8. Scope of delivery


- Rollator Fixi / Fox
 - o User manual (02.05.012)
- The reprocessing and maintenance instructions can be ordered or viewed and downloaded at www.rebotec.de/downloads.

9. Setting up for use

- The product has been carefully inspected in the factory for freedom from defects and completeness. Please check the product for any possible damage which may have occurred during transport and for completeness of the scope of delivery upon receipt. Assembly or adjustment shall be carried out by the specialist dealer.

9.1. Unfolding

- Place the Rollator on its wheels and unfold it by pushing down the rear wheel bar. The folding lock must be completely engaged.

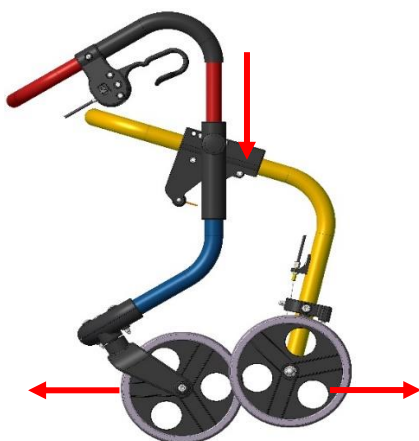

- Check that the Rollator is fully unfolded, the folding lock is engaged and the bag is properly secured.

9.2. Adjusting the handle height

- The height of the handlebar must be adjusted to your body height.



- Loosen the two star knob screws (1) and pull them out. Move the handlebar (2) to the desired height. The holes of the two tubes must be aligned on top of one another. Then tighten the star knob screw again.

- Check the function of the handbrake. (See 11.4, 11.6 & 13.2)
- Check that there are no kinks in the brake cables.
- Check that the seat board can be folded down easily.

10. Product data

- We reserve the right to make technical developments and improvements with respect to the design shown in this User Manual.
- See 15.

Characteristic	Value	
1. User weight	max. 100 kg	
FIXI		
2. Product weight*	6.2 kg	
3. Total weight*	8.2 kg	
FOX		
2. Product weight*	6.2 kg	
3. Total weight*	8.2 kg	
1. User weight	max. 100 kg	
FIXI with seat board		
2. Product weight*	6.9 kg	
3. Total weight*	8.9 kg	
FOX with seat board		
2. Product weight*	6.9 kg	
3. Total weight*	8.9 kg	

*Without accessories.

11. Operation of the product

- Before use, it must be ensured that the product is in proper condition. If defects or malfunctions are

evident, the product must not be used. Have the defects corrected by a specialist dealer before use.

11.1. Prepare for operation

- The product must be unfolded and the latches engaged. Check that the handbrakes are securely in place and that they function properly (see: 9.1 & 11.4).

11.2. Sitting down and standing up

- Before sitting down and standing up, make sure that the product is securely positioned to prevent it from slipping or tipping over. Apply the parking brake on both sides.
- If you are unable to sit down and stand up on your own, ask for help.

Sitting down

- Position yourself as close as possible to the seat board. Support yourself with your hands on the handlebar when sitting down.
- Make sure that you sit as far as possible on the seat surface and not on the edge, so that there is no risk of slipping.


Getting up

- Before standing up, make sure your feet are on the floor.
- Support yourself with your hands on the handlebar when standing up. Stand up slowly.
- Do not lean your body too far forward to avoid a potential risk of falling.

11.3. Pushing the product

- Support yourself on the handles while walking.
- Always walk as upright as possible and inside the frame. You can create more space within the frame by folding the seat board to form a tray.
- Do not lean too far over the handlebar.
- Only walk on level paths and avoid obstacles such as curbs and doorsteps.
- Avoid slippery and loose floor coverings.
- While walking, both brakes must be released.

11.4. Braking system

 *Improper use of the brake can cause unsafe situations and damage (increased wear, limited function).*


The product should not be moved if the brakes are applied or the wheels are locked.

Parking brake (1)

- Press down the two brake levers in area (A) until they engage. The two rear wheels are now braked. To release the brake, pull the brake levers back up in area (A).

Handbrake (2)

- Pulling up and holding both brake levers brakes both rear wheels. Releasing the brake lever allows the two rear wheels to move freely again.

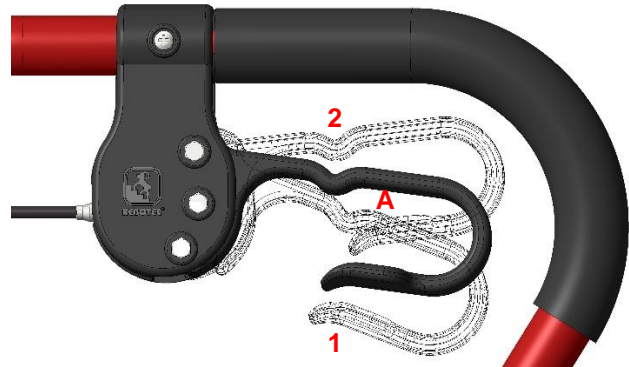


11.5. Folding of the product

Unfolding

- Place the product on its wheels and press down the frame. The folding lock must be completely engaged.
- See 9.1.

Folding

- Fold down the seat board and pull the pull cord (1) upwards with one hand until the lock is released.

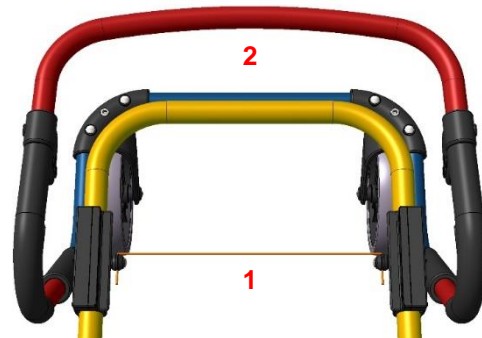


With the other hand, pull the handlebar (2) towards you.

11.6. Adjusting the brakes

- If the braking effect of the brakes begins to deteriorate, the product may no longer be put into operation and must be repaired by a specialist dealer. After inspection, they will decide whether readjustment is sufficient or whether parts have to be replaced.
- To adjust the brakes, loosen the lock nut (1) and then adjust the brake with the adjusting screw (2). Make sure that the wheels turn freely when the

brake is released and that the wheels are locked when the brake is applied. Re-tighten the lock nut.

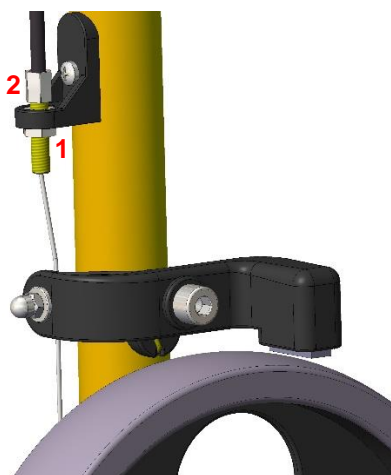

12. Troubleshooting

Fault	Potential cause	Remedy
Wheels turn sluggishly	Deposits	Cleaning, checking regularly
Engagement bolts are jammed	Deposits	Cleaning, replace engagement bolts
No or reduced braking function	Brake misaligned, brake lining worn, defective	Maintenance! Adjust brake, have lining replaced
Wheel set bent	Hitting an obstacle	Maintenance! Replacement
Visible cracks or breaks	Various causes	Maintenance! Replacement

13. Care and maintenance

13.1. Cleaning and care



The product can be cleaned using a mild conventional cleaning and disinfecting agent. After cleaning, wipe the product with a clean, moist cloth and then wipe it dry with a dry cloth.

- Do not use scouring agents or cleaning agents with aggressive ingredients (acids, thinners), as well as hard cleaning objects (brushes), as otherwise the surfaces may be damaged or deposits may form on the materials.
- The product is suitable for manual cleaning and disinfection. Please observe the reprocessing instructions (02.12.155).

13.2. Maintenance

- Wear and tear that is not recognised in time and/or improper handling, as well as irregular or no maintenance may result in accidents.



*For safety reasons, the product must be professionally maintained at **least once a year**.*

- Check product for visible damage
 - (cracks, breakage, loose parts)
- Check stability before use

Maintenance/inspection scope

- Maintenance should include the following inspection work and should be documented.
- Visual inspection for damage, wear and corrosion of all frame parts such as:
 - Frame
 - Handlebar
 - Front wheels (replace in pairs)
 - Rear wheels (replace in pairs)
 - Brake cables
 - Brake linings (replace in pairs)
 - Manual switch with brake lever
 - Seat board
 - Back rest pad
 - Check all connecting elements (screws, nuts) for completeness, fit and function
 - Function of the folding mechanism
 - Function of the height adjustment mechanism
 - Function of the brakes
 - Braking when moving (handbrake)
 - Locking the brake wheels (parking brake)
- In the event of damage and/or repairs, contact your authorised dealer immediately.
- Never attempt **any** unauthorised modifications or conversions to the product. These may impair the safety and proper functioning of the device. **In such cases, REBOTEC will not assume any liability.**
- Only use OEM spare parts for repairs. See 6.
- After maintenance or repair work has been carried out, we recommend you clean the product before using it again.
- Please note section 7& 14.



-
- Please observe the maintenance instructions (02.11.149).

14.Spare parts & accessories




No.	Art. no.	Designation
1	288.00.45	Front wheel
2	288.00.46	Rear wheel
3	280.10.53	Brake lining
4	286.65.25	Seat board
5	853.06.40	Star knob, complete
-	280.10.48	Back rest pad
-	164.30.00	Walking aid holder set



Use only REBOTEC spare parts and accessories to ensure the proper and safe functioning of the device.

15.Drawing & dimensions

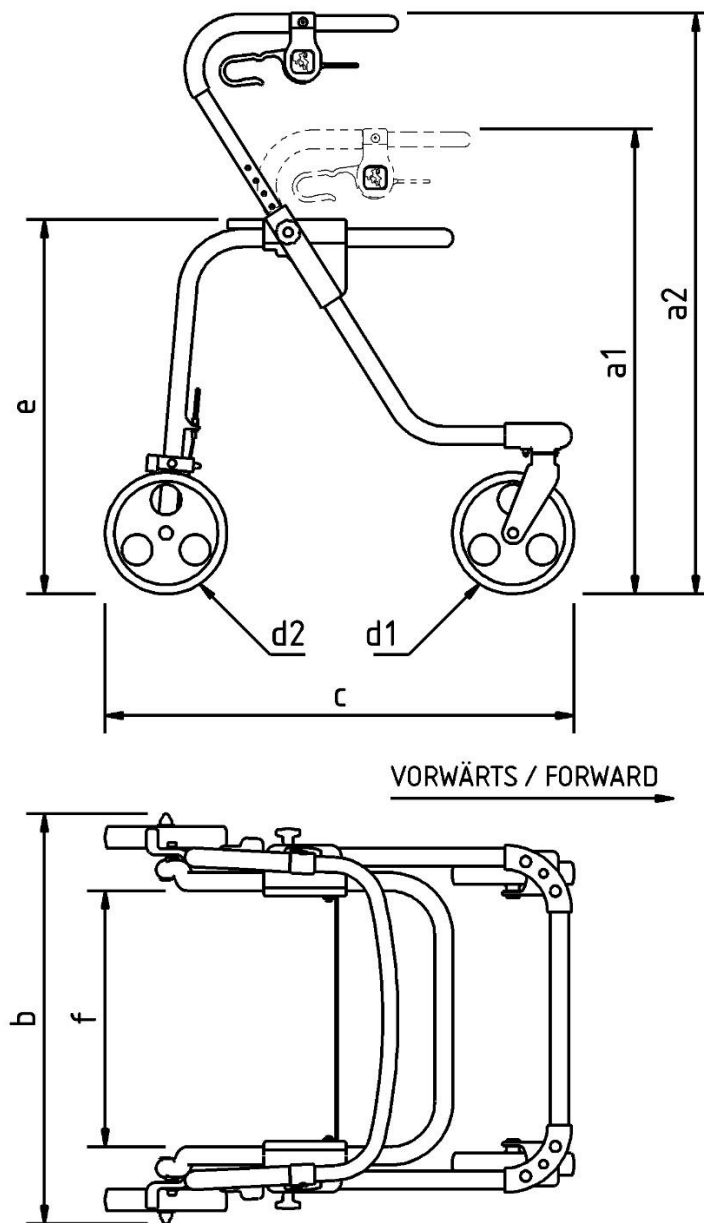

No.	Designation	FIXI / Dimensions in mm	FOX / Dimensions in mm
a1	Height min.	610	690
a2	Height max.	690	775
b	Width	585	585
c	Length	675	675
d1	Front castor diameter	200	200
d2	Rear castor diameter	200	200
e	Seat height	460	460
f	Seat width	380	380
bxc	Floor space	585 x 675	585 x 675



Dimensions and weights may vary depending on the equipment. There might be minor deviations due to production tolerances. Subject to change without notice.



REBOTEC® Rehabilitationsmittel GmbH

D-49610 Quakenbrück, Artlandstr. 57- 59

Telephone: +49 (0) 5431/9416-

Fax: +49 (0) 5431/9416- 66

Web: www.rebotec.de

Email: info@rebotec.de



Read this User Manual **before using the device for the first time** and keep it at hand for the user as well as for future use and subsequent sales.