



EN

BA-Nr.: 02.05.013/2022-03-A01



REBOTEC® Rehabilitationsmittel GmbH
DE-49610 Quakenbrück, Artlandstr. 57-59
Telephone: +49 (0) 5431/9416-0
Fax: +49 (0) 5431/9416-66
Web: www.rebotec.de
Email: info@rebotec.de



User Manual

Baby walker

Pluto / size 1		Pluto / size 2
REF	187.00.10	187.00.20
	60 kg	80 kg
Pluto / size 1 with direction lock		Pluto / size 2 with direction lock
REF	187.00.11	187.00.21
	60 kg	80 kg



Figure similar



Read this User Manual **before using the device for the first time** and keep it at hand for the user as well as for future use and subsequent sales.



Table of contents

1. Product description	3	13. Care and maintenance	8
1.1. Introduction	3	13.1. Cleaning and care	8
2. Features	3	13.2. Maintenance	9
2.1. Intended purpose	3	–(Cracks, breakage, loose parts).....	9
2.2. Conditions of use	3	14. Spare parts & accessories	10
2.3. Conditions of use	3	15. Drawing & dimensions	11
2.4. Storage & transport.....	3		
2.5. Equipment & functions	4		
2.6. Packaging	4		
2.7. Disposal	4		
3. Symbols & nameplates	4		
4. Safety instructions	4		
5. Incident reporting.....	5		
6. Warranty.....	5		
6.1. Complaint.....	5		
7. Reuse, lifetime	5		
8. Scope of delivery.....	5		
9. Setting up for use.....	6		
9.1. Unfolding.....	6		
9.2. Adjusting the handle height.....	6		
9.3. Adjusting the handles.....	6		
9.4. Deactivating the reverse lock	7		
9.5. Activation of the reverse lock	7		
9.6. Direction lock.....	7		
9.7. Mounting the saddle support unit (optional)	7		
9.8. Adjusting the saddle seat height.....	7		
10. Product data	8		
11. Operation of the product	8		
11.1. Use of the product.....	8		
11.2. Folding of the product	8		
12. Troubleshooting	8		

1. Product description

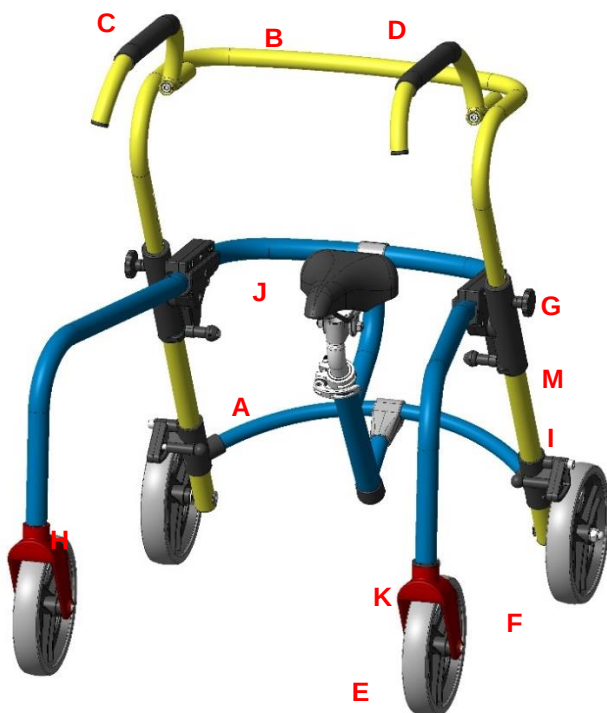


Figure shows:

A	Rear wheel bracket
B	Handlebar
C	Handle left
D	Handle right
E	Front wheel
F	Rear wheel
G	Star knob nuts, height adjustment
H	Steering stop
I	Brake head reverse lock
J	Saddle seat
K	Steering fork
M	Nameplate

1.1. Introduction

- The product described here is a medical device of risk class I according to **(EU) 2017/745 MDR**.
- In this user manual, you will find information on the safe and proper initial operation, use, maintenance and care of this product.
- Carefully read this user manual before using the device. Follow the safety instructions and act accordingly.

2. Features

2.1. Intended purpose

- The product is designed to promote and maintain mobility with regard to walking and running. A partial to complete relief of the lower extremities can be achieved. It enables the user to move around indoors independently or with the help of an attendant. There must be sufficient upper body musculature for safe use. The article does not impose any special requirements on the education of the user. The product can be equipped with optional accessories such as seat harness or saddle seats.

Specification:

Pluto / size 1

- For children
- Maximum user weight 60 kg
- Indoor use

Pluto / size 2

- For children
- Maximum user weight 80 kg
- Indoor use

2.2. Conditions of use

- Temperature range: 5 °C to 35 °C; direct radiant heat on the product must be avoided.
- Relative humidity: 40% to 75%. Outside of these conditions, the product must only be used for a short time.

2.3. Conditions of use

- Temperature range: 5 °C to 35 °C; direct radiant heat on the product must be avoided.
- Relative humidity: 40% to 75%. Outside of these conditions, the product must only be used for a short time.
- Folding for transport or storage is possible.
- This product may not be used to transport heavy objects.
- There must be a flat surface of sufficient size for the hoist to be placed on. See 15.

2.4. Storage & transport

- Temperature range: 5 °C to 25 °C; direct radiant heat on the product must be avoided.
- Store the product in dry rooms with a humidity of up to 65%.

2.5. Equipment & functions

- Foldable (without saddle seat)
- Handle height adjustable (see 9.2 & 15)

2.6. Packaging

- We recommend you keep the packaging for a later transport.
- Dimensions **Pluto / size 1**: 700 mm x 470 mm x 740 mm
- Dimensions **Pluto / size 2**: 700 mm x 470 mm x 840 mm

2.7. Disposal

- Disposal and recycling of used products and packaging must be carried out in accordance with currently valid regulations. Please contact a disposal company for further information.
- The Infection Protection Act must be observed.
- Please observe any labels and information on packaging material and act accordingly.

3. Symbols & nameplates

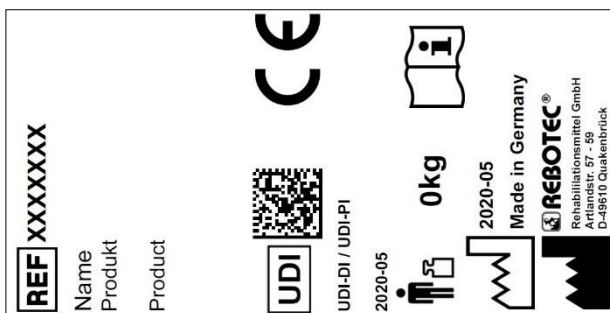


Figure similar.



The nameplate must not be removed!

For ordering spare parts and processing complaints, the device name, serial number and year of manufacture will be required.

Symbol	Meaning	Symbol	Meaning
	Read the manual!		Year of manufacture of Device
SN	Serial number		Important information!
REF	Article number		Maximum user weight in kg
	Care instructions		Maintenance instructions

Symbol	Meaning	Symbol	Meaning
	Applicable for indoor use only.		Manufacturer
	Visual signal		Language
	Read the User Manual!		Do not dispose of as household waste.
CE	Product complies with (EU) 2017/745 MDR		
	Caution! Observe the safety instructions		
UDI	Unique Device Identifier (Data: GTIN / LOT (SN) / year of manufacture)		
MD	Medical device label This product is a medical device.		

4. Safety instructions



Please observe the following safety instructions:

- Before using the device for the first time, the User Manual must be read completely in order to avoid hazards or damage caused by maloperation.
- Before each use, check the device for visible damage or deficiencies.
- In case of malfunctions or damage of any kind, the device must not be used.

Caution: Danger of falling!

- Be sure to check the following aspects before use:
 - Stability of the product.
 - Function of the reverse lock
 - Proper locking of the folding mechanism. (Without saddle seat)
- Only use the product if completely assembled.
- Do not slump carelessly on the product.
- Only use the height adjustment in the holes provided for it.
- Avoid using the product on slippery or uneven ground.
- Make sure that the reverse lock is always in locked position before using the product. Only release the two brakes after you seen you are able to stand stably.



- Do not start walking until you have grasped the handles with both hands.
- Users who are prone to uncontrolled actions or movements must be supervised when using the product.
- Use the article according to your mobility.
- Always walk within the baby walker as otherwise there is an increased risk of falling.

Caution: Danger of slipping!

- Do not grasp the handles with wet hands, as you could slip and lose your grip.
- When sitting on the saddle seat, always take up the entire seat surface.

Caution: Danger of tipping over!

- Avoid walking on steep inclines or declines; otherwise you could lose control of the product.
- Do not use the product on stairs or steps.
- Do not hang any objects on the product.

Caution: Danger of breakage!

- Observe the permitted user weight (2.1).
- The product is not a means of transport for heavy objects.
- The product is not suitable for passenger transport.
- Do not expose the product to great temperature fluctuations (2.2 & 2.4).

Caution: Danger of pinching or catching!

- There is a risk of pinching or catching on all moving parts of the product. Be especially careful when unfolding and folding that no body parts or objects are caught between the moving parts.

Caution: Risk of injury!

- There is a risk of injury with surface temperatures > 40 °C. Therefore, do not expose the product to high temperatures (e.g. sunlight, radiators, hot water). Allow the product to cool down before use.



If the product is used by patients with insensitive skin (no temperature perception or skin damage), a heat check (e.g. touching with the back of the hand) must be carried out by the nursing staff.

5. Incident reporting

- Serious incidents in connection with the product must be reported immediately to the manufacturer and BfArM.
 - BfArM: www.bfarm.de/DE

- Use the reporting functions and provided forms.

◦ Manufacturer: pms@rebotec.de

6. Warranty

- For this product the manufacturer grants a warranty of 12 months. Compliance with the general terms and conditions(www.rebotec.de/agbs), as well as the intended use, are a basic prerequisite. The statutory warranty provisions shall apply.
- Unauthorised modifications to this product will void the product's conformity and the warranty.

6.1. Complaint

- Please contact us before returning the product.
- To reduce transport damage, use the original packaging if possible.
- The Infection Protection Act must be complied with.
 - Please make sure that the product does not pose a risk of infection when returned.
 - Enclose the supplied information sheet on safety with the product.
 - Costs may incur for returns that are not marked non-objectionable.

7. Reuse, lifetime

- The product is suitable for reuse. How often the product can be reused depends on the frequency and nature of use.
- Before reuse, the product must be reprocessed hygienically and the technical and functional safety and reliability of the product must be checked and restored if necessary. See 13.
 - For more detailed information, please see the reprocessing instructions (02.12.155) and the maintenance instructions (02.11.149).
 - The documents can be downloaded from the following link.
 - www.rebotec.de/downloads
- When used as intended, the product has a lifetime of 3 years. The effective lifetime may vary depending on the frequency and intensity of use and the general condition.

8. Scope of delivery

- Pluto / size 1-Pluto / size 2
 - User manual (02.05.013)


- o Allen key 6 mm
- o Saddle seat (accessory)
- The reprocessing and maintenance instructions can be ordered or viewed and downloaded at www.rebotec.de/downloads.

9. Setting up for use


- The product has been carefully inspected in the factory for freedom from defects and completeness. Please check the product for any possible damage which may have occurred during transport and for completeness of the scope of delivery upon receipt. Assembly or adjustment shall be carried out by the specialist dealer.

9.1. Unfolding

- Place the item on the front wheels and pull the rear wheel bracket upwards.
- The baby walker locks automatically in its final position.

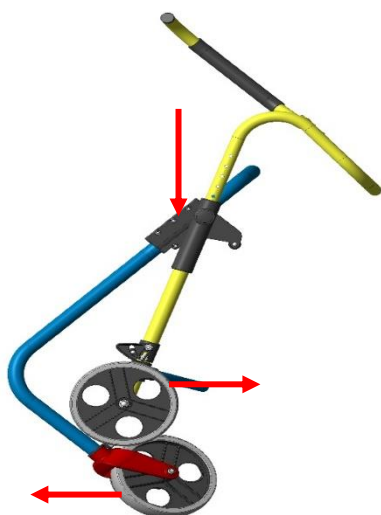


- Check that the item is fully unfolded and the folding lock is engaged.

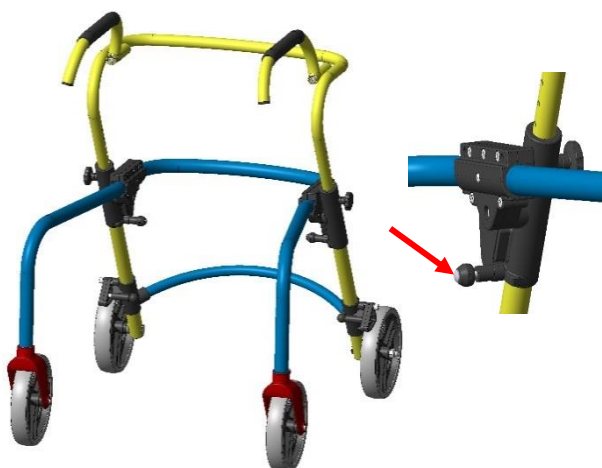


9.2. Adjusting the handle height

- The height of the handlebar must be adjusted to your body height.
- Loosen the two star knob screws (1) and pull them out. Move the handlebar (2) to the desired height. The holes of the two tubes must be aligned on top of one another. Then tighten the star knob screw again.

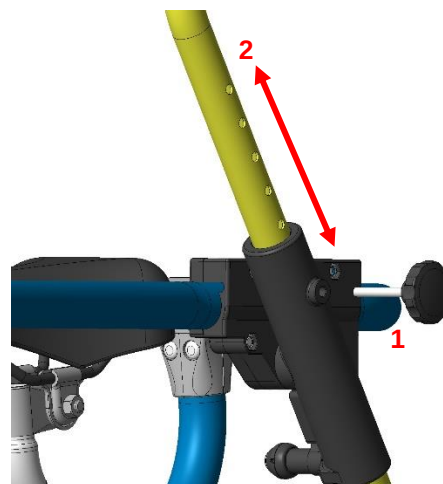


9.3. Adjusting the handles

- Insert the two handles (1) sideways into the inside opening (2) of the handle bracket. Adjust the handles to the desired grip position. Be careful not to pull the handle out of the holder (2). The handle must be in contact with the stop. Then, firmly tighten the screws (1) with an Allen key.

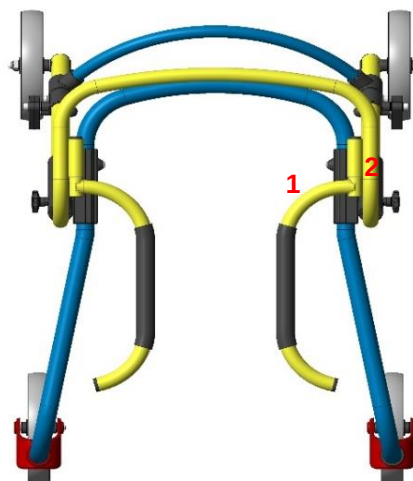


9.4. Deactivating the reverse lock

- To deactivate the reverse lock, pull the snapper bolt (1) towards the interior of the unit and turn the brake (2), then let the snapper bolt (1) engage in the straight groove.

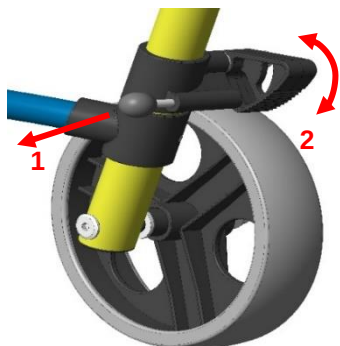


9.5. Activation of the reverse lock

- To activate the reverse lock, pull the snapper bolt (1) towards the interior of the unit and turn the brake (2) clockwise a little onto the tyre, then let the snapper bolt (1) engage in the curved groove.
- Then check the function of the reverse lock

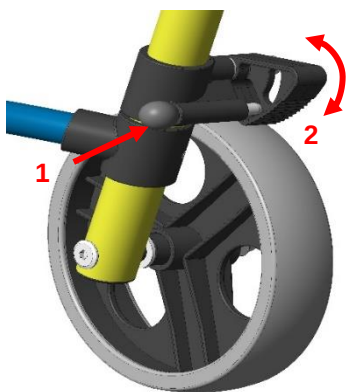


9.6. Direction lock

- If the front wheels are equipped with a direction lock, the steering stop is activated when the pin is engaged and the wheel forks can no longer turn.

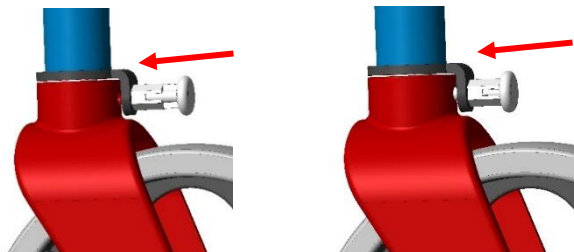

- By pulling out and turning the bolt by 90 degrees, the steering stop is deactivated and the wheel forks with the wheel can be turned by 360 degrees.


9.7. Mounting the saddle support unit (optional)

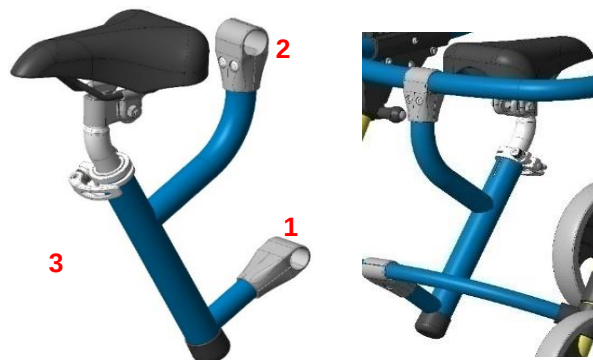


- Remove the screws and insert the half-shell of the connecting element (1) centrally onto the cross strut.
- Reinsert the screws.
- Insert the support tube in pos.1 and tighten screws slightly.
- It must be possible to move the connection at this stage.
- Proceed in the same way with the second connector (2).
- Tighten all four bolts and place the saddle support in the support tube (3).

9.8. Adjusting the saddle seat height

- If a saddle seat unit is mounted on the baby walker, there is the option of steplessly adjusting the saddle seat height.

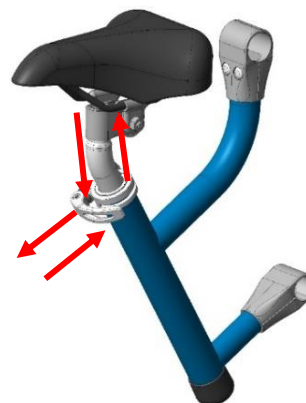

- To adjust the saddle seat height, loosen the quick release on the saddle mounting tube. Adjust the saddle to the required height (pull out the saddle tube). Align the saddle seat parallel to the frame and clamp the saddle tube again with the quick release clamp.

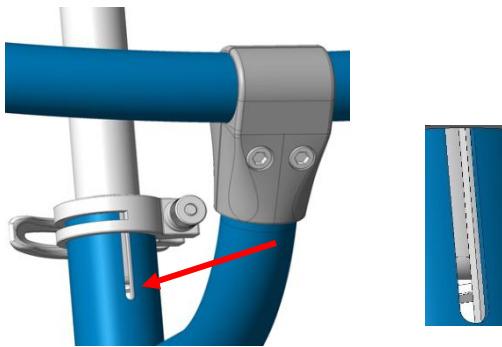

Max. adjustment height is reached when the end of the saddle seat tube becomes visible in the slot.

10. Product data

- We reserve the right to make technical developments and improvements with respect to the design shown in this User Manual.
- See 15.

Characteristic		Value
Pluto / size 1		
1.	Product weight*	8.0 kg
2.	Total weight*	10.0 kg
Pluto / size 2		
1.	Product weight*	8.2 kg
2.	Total weight*	10.2 kg

*Without accessories.

11. Operation of the product

- Before use, it must be ensured that the product is in proper condition. If defects or malfunctions are evident, the product must not be used. Have the defects corrected by a specialist dealer before use.

11.1. Use of the product



Observe the safety instructions

- Support yourself on the handles while walking.
- Only use level paths and avoid obstacles such as curbs and doorsteps.
- Avoid slippery and loose floor coverings.
- While using the baby walker, both brakes must be released.

11.2. Folding of the product


Folding of the product is possible only without the saddle seat.

Unfolding

- Place the product on its wheels and press down the frame. The folding lock must be completely engaged.
- See 9.1.

Folding

- Pull out the two snap locks (A) completely and pull the tube bracket (B) towards you.

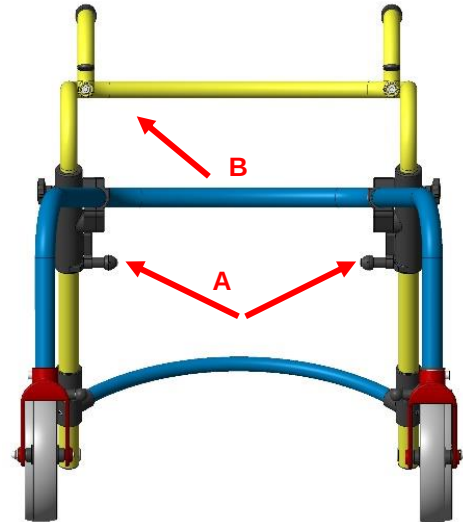

12. Troubleshooting

Fault	Potential cause	Remedy
Wheels turn sluggishly	Deposits	Cleaning, checking regularly
Locking bolts are jammed	Deposits	Cleaning, Replace the locking bolt
No or reduced braking function	Brake worn, defective	maintenance! Have castors replaced
Wheel set bent, broken	Hitting an obstacle	maintenance! Replacement
Visible cracks or breaks	Various causes	maintenance! Replacement
Steering stop	Deposits	Regular check, replacing the steering stop
Slip resistance reduced	Worn foot capsules	maintenance! Replacement
	Wetness	Dry foot capsules

13. Care and maintenance

13.1. Cleaning and care



The product can be cleaned using a mild conventional cleaning and disinfecting



agent. After cleaning, wipe the product with a clean, moist cloth and then wipe it dry with a dry cloth.

- Do not use scouring agents or cleaning agents with aggressive ingredients (acids, thinners), as well as hard cleaning objects (brushes), as otherwise the surfaces may be damaged or deposits may form on the materials.
- The product is suitable for manual cleaning and disinfection. Please observe the reprocessing instructions (02.12.155).

13.2. Maintenance

- Wear and tear that is not recognised in time and/or improper handling, as well as irregular or no maintenance may result in accidents.



*For safety reasons, the product must be professionally maintained at **least once a year**.*

- Check product for visible damage
 - (Cracks, breakage, loose parts)
- Check stability before use

Maintenance/inspection scope

- Maintenance should include the following inspection work and should be documented.
- Visual inspection for damage, wear and corrosion of all frame parts such as:

- Handlebar
- Front wheels (replace in pairs)
- Check all connecting elements (screws, nuts) for completeness, fit and function
- Function of the folding mechanism
- Function of the height adjustment mechanism
- Function of the brakes (parking brake)
- In the event of damage and/or repairs, contact your authorised dealer immediately.
- Never attempt **any** unauthorised modifications or conversions to the product. These may impair the safety and proper functioning of the device. **In such cases, REBOTEC will not assume any liability.**
- Only use OEM spare parts for repairs. See 6.
- After maintenance or repair work has been carried out, we recommend you clean the product before using it again.
- Please note section 7 & 14.
- Please observe the maintenance instructions (02.11.149).

14.Spare parts & accessories




No.	Art. no.	Designation
1	288.00.45	Front wheel
2	288.00.46	Rear wheel
3	287.10.44	Steering stop
4a	287.20.10	Seat unit complete, size 1
4b	287.20.20	Seat unit complete, size 2
5	853.06.40	Star knob, complete
6	446.10.60	Brake head reverse lock
-	280.10.48	Universal backrest pad
-	02.05.013	User Manual
-	02.11.149	Maintenance Instructions
-	02.12.155	Reprocessing Instructions



Use only REBOTEC spare parts and accessories to ensure the proper and safe functioning of the device.

15.Drawing & dimensions

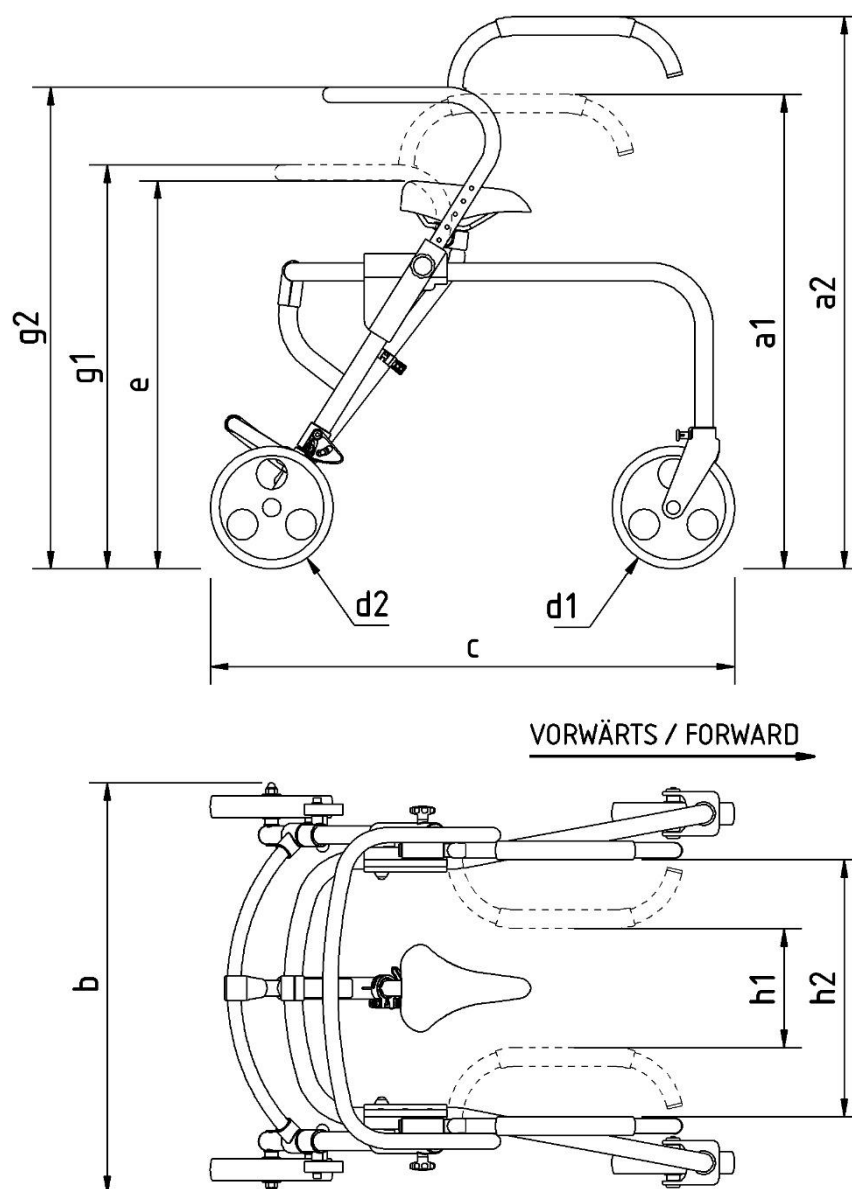

No.	Designation	Size 1 / dimensions in mm	Size 2 / dimensions in mm
a1	Height min.	460	540
a2	Height max.	800	890
b	Width	650	670
c	Length	800	860
d1	Front castor diameter	200	200
d2	Rear castor diameter	200	200
e	Seat height	460-580	510-630
h1	Inner width -handles	180	200
h2	Outer width -handles	380	410
g1	Height -backrest min.	580	660
g2	Height - backrest max.	710	790



Dimensions and weights may vary depending on the equipment. There might be minor deviations due to production tolerances. Subject to change without notice.



REBOTEC® Rehabilitationsmittel GmbH
D-49610 Quakenbrück, Artlandstr. 57-59
Telephone: +49 (0) 5431/9416-0
Fax: +49 (0) 5431/9416-66
Web: www.rebotec.de
Email: info@rebotec.de



Read this User Manual **before using the device for the first time** and keep it at hand for the user as well as for future use and subsequent sales.